

# SIRONA CEREC STL EXPORT INSTRUCTIONS

## Sirona Cerec System Requirements:

- CEREC Software 4.5.x
- STL Open Scan Export License key -  
activated using the license manager

If your system is not updated or requires the STL license key please contact  
Sirona directly on: 1300 747 662(AU) | 0800 747 662 (NZ)




## STL EXPORT

To generate an STL from your scan and upload to Race Dental, please follow the below steps

\*Note: Only scan data is exported. CAD data such as restorations, preparations margins, insertion axis or implant positions are not exported.


Go to the System menu in the top  
left-hand corner & click on Export.

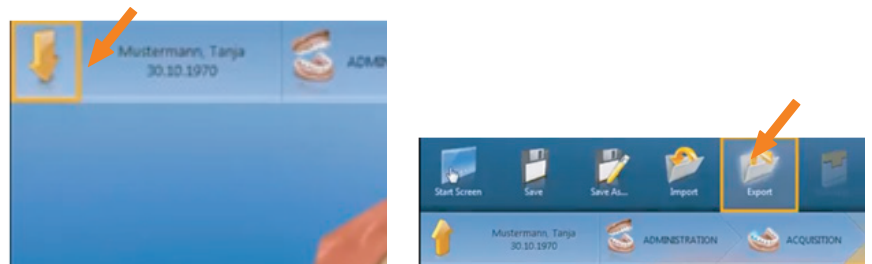

A save window will appear. Choose  
the location you wish to save the Scan  
(e.g. Desktop) and change the name if  
required in the box on the right.



Click the drop down to select the High  
Resolution STL and click Save.

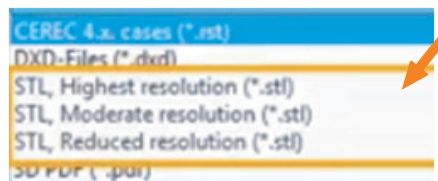

Two files, An Upper Jaw and Lower Jaw STL will now be saved in the location you selected above. Please upload the files  
into Race Dental's Secure Upload Portal <https://upload.racedental.co.nz/> Turn over for the Upload Portal instructions.

# RACE DENTAL SECURE UPLOAD PORTAL

For STL scans, DSD Photographs, Patient Data or other files

Open your browser and navigate to  
<https://upload.racedental.co.nz/>



Enter your email address and password or create a new account for your practice. If you do not already have one, select "Create One Here".

## User Login

Email Address

Password

Login

Don't have an account? [Create one here](#)  
Forgot password? [Reset it here](#)

Complete new account details for your clinic, if you have not already done so.

Welcome to the case sending portal. If you already have an account, you can log in [here](#), otherwise fill in the following fields

Name	<input type="text"/>
Race Dental Account Number	<input type="text"/>
Email Address	<input type="text"/>
Clinic	<input type="text"/>
Suburb	<input type="text"/>
Password	<input type="password"/>
Confirm password	<input type="password"/>
<input type="checkbox"/> I'm not a robot	
<input type="button" value="Submit"/>	

Enter all the necessary details for your case as shown (patient name, material, shade, special requests, etc).

You may prefer to scan or take a photo of a Race Dental lab sheet if it is easier. Click on the "Select Files" button and select all the files that you want to send with your case (e.g. upper and lower STL, shade photos, DSD photos, scanned lab sheet etc).

Patient name / case ID:	<input type="text"/>
Case type:	<input type="checkbox"/> Crown & Bridge <input type="checkbox"/> Implants <input type="checkbox"/> Acrylic <input type="checkbox"/> Orthodontics <input type="checkbox"/> Orthognathics <input type="checkbox"/> DSD
Description / notes:	<input type="text"/>
Requested due date	<input type="text"/>
<i>Laboratory will confirm if due date is achievable.</i>	
Attach files:	<input type="button" value="+ Select files"/>
<i>Please upload STL photos and/or lab sheet(s) as appropriate.</i>	
<input type="button" value="Submit"/>	

When you are satisfied that all information necessary to complete the case has been provided, click "Submit". A Race Dental customer service representative will contact your clinic if any issues/concerns are encountered or if further information is required.

If you require any further assistance from Race Dental, please contact our support team on **(09) 887 0327** or email [customersupport@racedental.co.nz](mailto:customersupport@racedental.co.nz)