

HOW TO EXPORT SCANS AS STL FORMATS FROM iTERO SCANNERS

You cannot export directly from the machine and instead should use a computer



1. In your web browser, type **www.myitero.com** and log into your account.

2. Once logged in, you will see a list of your patients scanned and sent from your iTero scanner.

3. Navigate to the correct patient and click on their ID #. This will take you to the order information page.

4. Click on **"Export" (OrthoCAD 3.5 or higher)**. Now a study model will pop up and a box that says **"OrthoCAD Export"**. This will open the files and allow you to download them.

Order Information Search Order ID Save Cancel

Order ID:	39487787	First Name:	*	Company:	
Order Code:	01CSG9K0	Last Name:	*	Doctor:	
Case Type:	iRecord	Chart Number:		Doctor License:	131165
Registration Date:	2020/07/28	Acquisition Date:	2020/07/28	Tracking Number:	
State:	Completed	Modelling Center:		Order Status:	Active

Rx notes:

Actions: [Open File Export \(OrthoCAD 3.5 or higher\)](#) [Change Order](#) [Image Management](#) [Open BX](#)

Order Details


Order Details ID	Item	Quantity	Ship To	Bill To	Due Date	Shipping Date	Delivery Date	Status
55732069	iRecord	1.00			2020/07/28			Completed

5. In the **OrthoCAD Export box**, select the setting for download. Choose the following:

Export type: Choose the “Open Shell” option.

Data Format: Choose “Two files” (arches oriented in occlusion).

Folder Name: We recommend changing this number to your patient’s name.

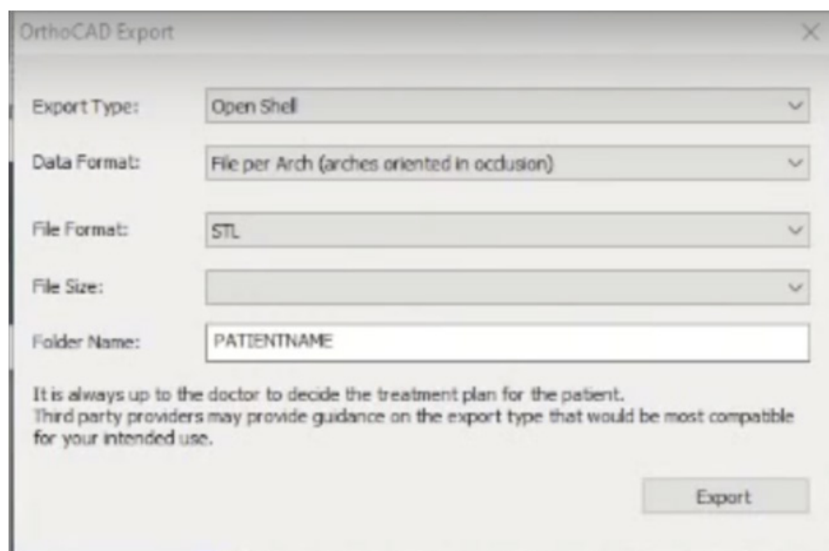

6. Click on **"Export" (OrthoCAD 3.5 or higher)**. Now a study model will pop up and a box that says **"OrthoCAD Export"**. This will open the files and allow you to download them.